

LENS CARE PLAYS A BIG ROLE

in keeping contact lenses fresh,
clean and comfortable.

For contact lens comfort, experience
the deep clean of CLEAR CARE®
Cleaning and Disinfecting Solution—
your **one-bottle solution for cleaning
and disinfecting.**

With powerful cleaning action in its
unique lens case, CLEAR CARE®
Solution provides a **natural, no-lens feel**
for today's soft contact lens wearers.

**If you are sensitive to preservatives
found in other contact lens solutions,
try CLEAR CARE® Solution.**



Powerful TRIPLE-ACTION CLEANING

With **3% HYDROGEN PEROXIDE**,
CLEAR CARE® Solution delivers powerful
TRIPLE-ACTION CLEANING, to give you an
outstanding lens-wearing experience.



DEEP CLEANS




LOOSENS DIRT



ENHANCES
PROTEIN REMOVAL

Get exceptional cleaning and outstanding
disinfection, plus a unique lens case
designed specifically for CLEAR CARE® Solution.

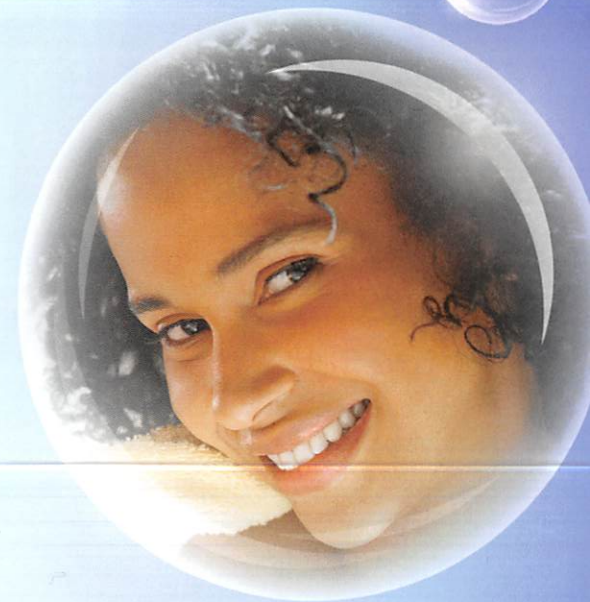

Powerful, for a difference you can see.

With its bubbling action, you can actually see
CLEAR CARE® Solution working!

Gentle*, for a difference you can feel.

After 6 hours, the unique CLEAR CARE® Solution
disposable lens case, included with each
purchase, neutralizes CLEAR CARE® Solution
to create a gentle* saline solution, much like
tears—without preservatives. No final saline
rinse is required.

Your contact lenses + CLEAR CARE® Solution = comfort you can count on.

AS EASY AS 1-2-3

Just follow these 3 simple steps:**



1. Rinse

Place lenses in basket holder and rinse with CLEAR CARE® Cleaning and Disinfecting Solution for 5 seconds.



1. Rinse

Fill case to "fill line" with CLEAR CARE® Solution.



3. Soak

Store lenses in case 6 hours or overnight.
DO NOT REMOVE BEFORE 6 HOURS.

IMPORTANT TIPS

Using CLEAR CARE® Solution is easy, but be sure to remember these four important tips.



Never use with a flat lens case.
CLEAR CARE® Solution must be used with the special lens case provided.



Because the **solution needs time to neutralize**, never remove lenses from the case until at least 6 hours later.



Never rinse lenses with CLEAR CARE® Solution prior to inserting lenses into your eyes. If you want to rinse lenses, use a sterile saline solution.



Never squirt CLEAR CARE® Solution directly into your eyes.

Visit clearcaresolution.com to learn more.



Alcon



Comfort You Can
COUNT ON



*When used as directed. **Eye care practitioner instructions may differ. See package insert for complete instructions.

© 2011 Novartis 11/11 CCS12000PE