




3 EASY STEPS

to Fitting the Ocutech® SightScope® FLIP


1

Align the telescope eyepiece(s) with the pupil(s) of the eye(s) that will be using the telescope.

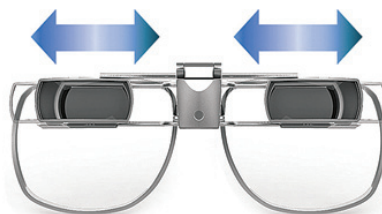


2

Adjust the frame nose-pads to position the vertical position of the eyepiece(s).

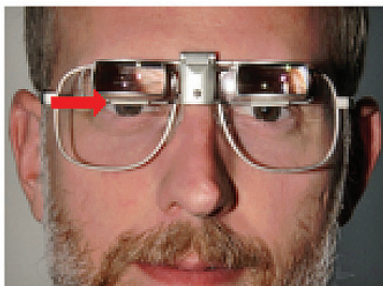

FOR BIOPTIC POSITION:
*align the bottom of the eyepiece
with the top of the pupil*

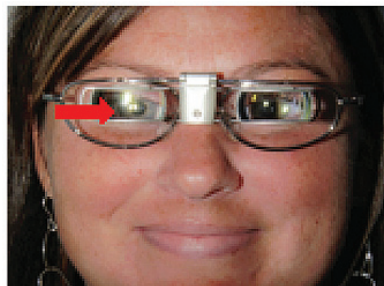


FOR FULL DIAMETER POSITION:
*position the eyepiece to
surround the pupil*

3


Using the Allen wrench tool adjust the viewing angle so that when looking through the telescope the line of sight of the user aligns with that of the telescope.



Using adjustment tool

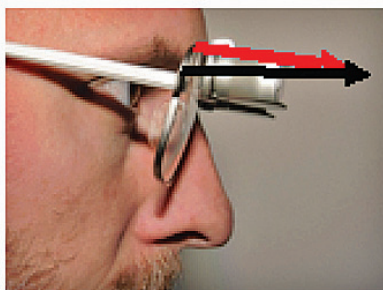

Wrong

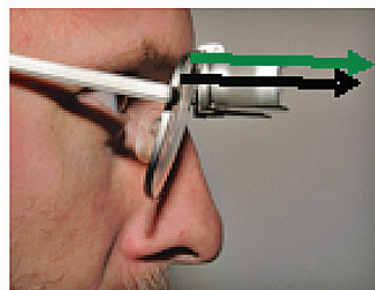

Correct

***And, don't forget to flip it out
of the way when not needed!***



OCUTECH Vision Enhancing Systems