

## ***Is an Ocutech Bioptic right for your patient?***

August 2019

Ocutech bioptics provide the widest field of view possible for the telescopic powers that they provide. They are designed to be easy to demonstrate, fit and prescribe, allowing the potential wearer to experience as close-to-final a visual benefit possible. The innovative mounting design allows the prescriber to easily fine-tune the fitting to maximize its performance and benefit to the user.

There are several factors to explore when considering the appropriateness of an Ocutech bioptic for an individual. We offer a list of factors you may choose to consider.

### **1. Vision**

- a. **BCVA in the better-seeing eye (hopefully the dominant eye) is 20/300 or better**
- b. **Field of view of at least 40 degrees diameter with regular glasses if used**
- c. **Absence of hemianopsia**

### **2. The prospective patient seeks to improve their distance and/or midrange vision for activities that might include:**

- a. Independent travel
- b. Classroom
- c. Signage
- d. Shopping
- e. Social engagement
- f. Table/desk activities
- g. Computer screens
- h. Driving
- i. Music
- j. Theater/movies
- k. Museums/galleries
- l. Hiking
- m. Gardening

### **3. They have promising manual dexterity**

### **4. Need for hands-free visual support**

- a. Either fixed distances with minimal need to focus the device
- b. Use of Autofocusing bioptic

### **5. Intact or adequate cognitive status**