

OCUTECH® SideSight™


SideSight Settings UserGuide.v4.1

Instructions to install and use the SideSight User Settings program:

This only works on a windows computer.

To Install the **SideSight User Settings** program:

1. Create a folder (on your desktop) labelled **SideSight User Settings** and download this file to this folder. This may take several minutes.

 [SideSightSettingsInstaller1.2.0.exe](#)


2. Once downloaded double-click on the SideSightSettingsInstaller.exe file. This will create the **SideSight User Settings Application** on your computer.

The Application Link will look like this:



To Update the SideSight Firmware:

The Control Panel will look like this:

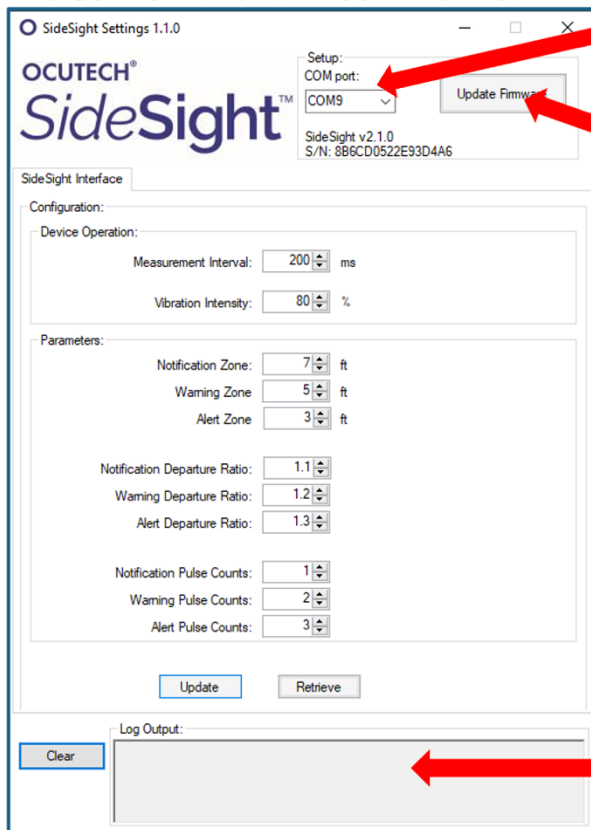

1

1. Open the SideSight Settings application

2

2. Connect a USB-C cable from the SideSight to your computer

3. Click the COM port selection at the top which will prompt the proper port automatically (#1)

4. Click update firmware button on the top right (#2)- a window will open automatically prompting you to select a ZIP file.

5. Double-click the **Firmware** folder and then click on the **SideSightFirmware2.2.0 ZIP** file and click 'open'.

The full path of the firmware ZIP file is:


C:\program Files (x86)\Ocutech\SideSight\Firmware\SideSightFirmware2.2.0.zip

6. The firmware will be updated automatically. You can follow the update progress in the bottom window (#3). Depending upon the speed of your computer this may take several seconds.

3

To Change the Settings on the SideSight

Once the Firmware has been updated, you do **not** have to re-run the firmware update to make changes to the SideSight settings.

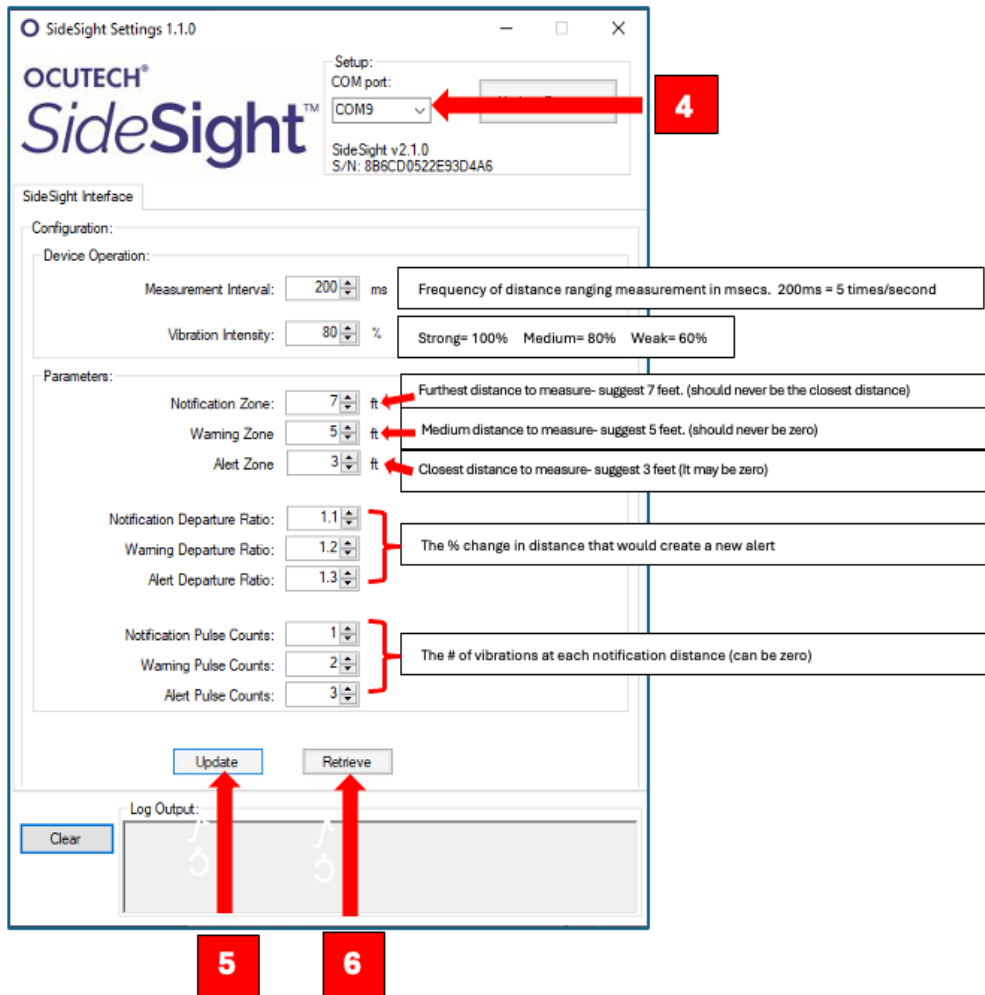

1. Connect a USB-C cable from the SideSight to your computer
2. Click on the COM port which will be automatically displayed. (#4)
3. The current SideSight settings will now be displayed.
4. Alter any of the SideSight settings as may be required.
5. Click the "Update" button on the bottom left. (#5)
6. The 'Retrieve' button reads the current settings back from the SideSight. (#6)
7. The updated settings will be stored permanently until you change them again.
8. To find the original configuration settings, start the SideSight Settings application without attaching the device and observe the configuration values displayed. These are the original default settings.