

## Instructions to install and use the SideSight™ User Settings program:

*This only works on a windows computer.*


To Install the SideSight™ User Settings program:

1. Create a folder (on your desktop) labelled SideSight User Settings and download this file to this folder. This may take several minutes



2. Once downloaded double-click on the SideSightSettingsInstaller.exe file. This will create the SideSight User Settings Application on your computer.

The Application Link will look like this:

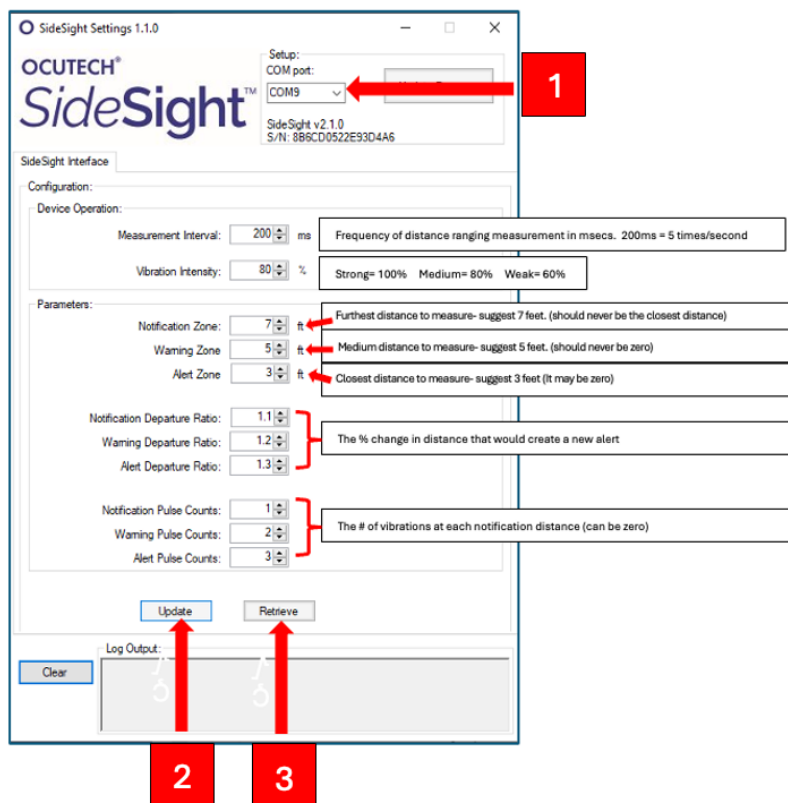

## To Change the Settings on the SideSight™


1. Connect a USB-C cable from the SideSight to your computer
2. Click on the COM port which will be automatically displayed. (#1)
3. The SideSight's current settings will now be displayed.
4. Alter any of the SideSight settings as you may prefer.
5. Click the "Update" button on the bottom left. (#2)
6. The 'Retrieve' button reads the current settings back from the SideSight. (#3)
7. The updated settings will be stored permanently until you change them again.

**NOTE:** To find the original configuration settings, start the SideSight Settings application without attaching the device and observe the configuration values displayed. These are the original default settings.

## To Update the SideSight™ Firmware:

If Ocutech releases an updated Firmware application you will need to update your current SideSight application.

Please follow these steps to update the SideSight Firmware Application.

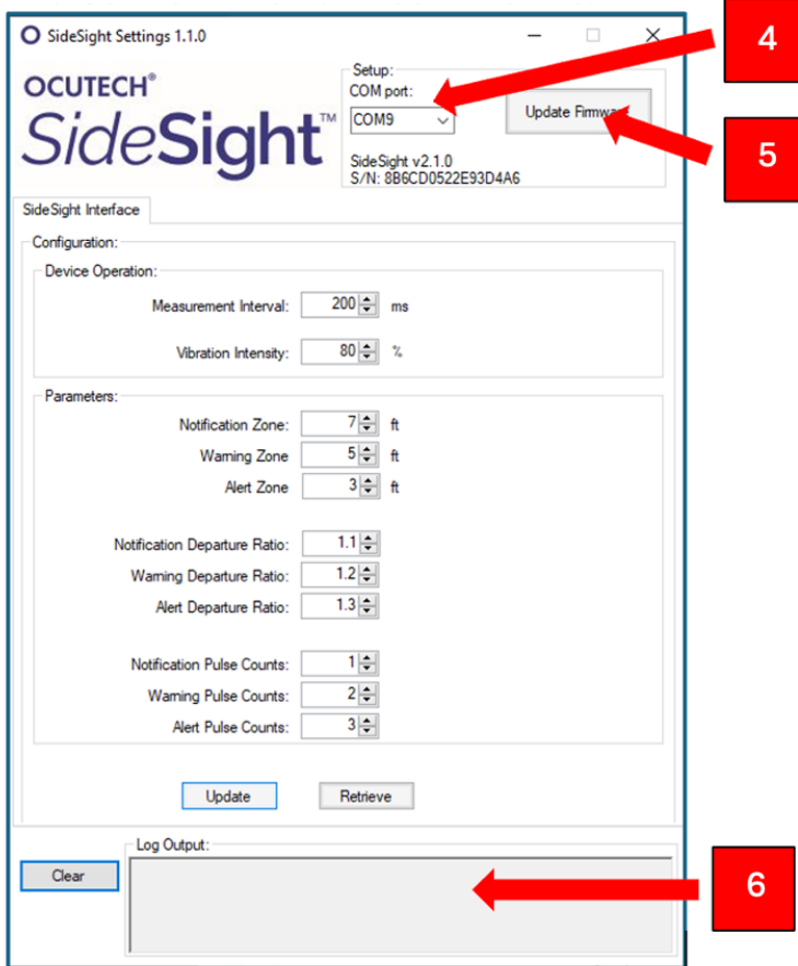

1. Open the SideSight Settings application
2. Connect a USB-C cable from the SideSight to your computer
3. Click the COM port selection at the top which will prompt the proper port automatically (#4)
4. Click update firmware button on the top right (#5)- a window will open automatically prompting you to select a ZIP file.
5. Double-click the **Firmware** folder and then click on the **SideSightFirmwarex.x.x ZIP** file and click 'open'. **DO NOT** un-zip the file.
6. The firmware will be updated automatically. You can follow the update progress in the bottom window (#6). Depending upon the speed of your computer this may take several seconds.